MED+CARE

Disinfecting WIPES

Kills 99.9% of
Bacteria & Viruses
(in 30 seconds/
up to 2 minutes)

Hydrogen Peroxide Cleaner Disinfectant Wipes

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- Bactericidal, Virucidal; Tuberculocidal
- Compatible with Equipment Surfaces
- Formulated with Hydrogen Peroxide Technology

- One-Step Disinfectant Cleaner
- No Harsh Chemical Fumes
- Non-Corrosive
- Non-Flammable

ACTIVE INGREDIENT:

Hydrogen Peroxide1.4%
Other Ingredients98.6%
TOTAL.....100.0%

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KEEP OUT OF REACH OF CHILDREN

CAUTION: See back panel for additional precautionary statements

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DIRECTIONS FOR USE: To open package, pull out wipe from center of roll. Next sheet pops up automatically. Close lid to retain moisture.

DISINFECTION: This product cleans and disinfects hard, nonporous medical surfaces in one step with no rinsing required.

TO CLEAN AND DISINFECT HARD, NONPOROUS SURFACES: Wipe surfaces until completely wet. Allow this product to remain wet for contact time listed on label. Let air dry. For heavily soiled areas, a precleaning is required.

PRECAUTIONARY STATEMENTS: HAZARDS TO HUMANS AND DOMESTIC ANIMALS.

CAUTION: Causes moderate eye irritation. Avoid contact with eyes or clothing. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco or using the toilet. **FIRST AID:** If in eyes: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing. Call the poison control center at 1-800-222-1222 or doctor for treatment advice. Have the product container or label with you when calling a poison control center or doctor or going for treatment.

Storage and Disposal: Store this product in a cool, dry area, away from direct sunlight and heat to avoid deterioration. Dispose of wipe in trash after use. Do not flush. Do not reuse or refill except as described in the directions for use. Refill only with this product. Recycle empty container or discard in trash.

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